

ULTRACOVER[®]

Transoesophageal Kit

A complete kit for the Transoesophageal Echocardiography (TOE) probes used in adult and pediatric patients.

The kit consists of an ULTRACOVER[®], a syringe filled with transmission gel, an extension tube, a bite guard and a fixing aid.




1. Remove the cap and fasten the connector to the syringe.



2. Slip the latex cover on the syringe and push the transmission gel into the tip of the cover.



3. Parcel the gel out.

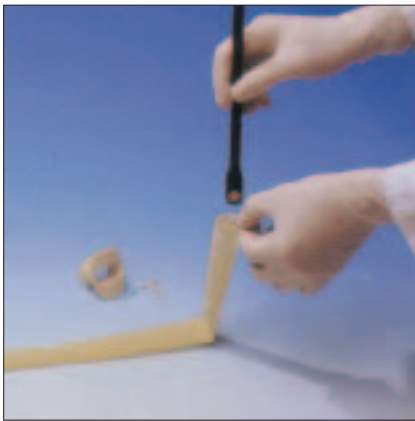

4. Slip the transducer into the latex cover.



5. Fasten the latex cover by means of the twistlock and apply the biteguard.



6. The probe is now ready for use.

Art.no.	Description	Selling unit
86590	TOE probe cover 25/11/1000, folded, sterile	10
86602	TOE probe cover 25/11/1000, folded	10
87602	TOE probe cover 25/11/1000 PU, folded, sterile	10
87106	TOE kit 25/11/1000 PU, folded, sterile bite guard, gel, lock, connector	12
86592	TOE kit 25/11/1000, folded, sterile bite guard, gel, lock, connector	12
86593	TOE kit pediatric, folded, sterile bite guard, gel, lock, connector	12
86599	TOE probe cover 32/10/1000, folded	10