

# Urinar

Disposable external male catheter

SIMPLE. DISCRETE.



# Urinar

Disposable external male catheter

The external catheter is simple to use, light and inconspicuous to carry.

## Instructions for use:

1. Choose the right size; 7 sizes are available as indicated on the packaging
2. Put the catheter on the glans and unroll the condom part; make sure that the opening of the urethra is in front of the urinar opening; it is possible to powder the penis in advance; in case of sensitive patients the external catheter can be treated with silicon spray on the inside
3. The outlet of urine can be controlled by connecting the catheter by means of a connection tube with a portable urine bag, which can be fixed to the leg.

**N.B.** Change the catheter regularly to prevent irritation of the penis.



## PACKAGING AND ORDERING INFORMATION

Selling unit: **10 pieces**

**Urinar size 0  
extra small (Ø 15mm) 6000**

**Urinar size 1  
very small (Ø 17mm) 6010**

**Urinar size 2  
small (Ø 19mm) 6020**

**Urinar size 3  
medium (Ø 22mm) 6030**

**Urinar size 4  
large (Ø 25mm) 6040**

**Urinar size 5  
very large (Ø 28mm) 6050**

**Urinar size 6  
extra large (Ø 34mm) 6060**

For more information on Microtek products or to place an order, call: **+31 (0)575 599200**

Visit our website: [www.microtekmed.com](http://www.microtekmed.com)

Art.no.	Description	Selling unit
<b>Fixation strip</b>		
Strip (15 x 3 cm) of soft foam with Velcro on one side.		
6095	Fixation strip for external catheter	10
<b>Uri-tape</b>		
Uri-tape is a self-adhesive foam. This foam does not stick to the skin, only to itself.		
6400	Uri-tape	2

**Ecolab Healthcare**  
Clean. Safe. Healthy.

**Microtek Medical B.V.**  
A Division of Ecolab  
Hekkehorst 24  
NL-7207 BN Zutphen

© 2010 Ecolab Inc. All rights reserved.

**Ecolab Europe GmbH**  
Richtistr. 7  
CH-8304 Wallisellen  
[www.ecolab.com](http://www.ecolab.com)

**ECOLAB®**