



TOE Cover

Trans Oesophageal Echocardiography Probecover

SAFE. HYGIENIC. LATEXFREE.

Unique latexfree and high elastic probe cover with applicator for easy use and transmission gel syringe already set in place.



TOE Cover

Trans Oesophageal Echocardiography Probecover

Our Offer	Your Benefit
Dipped probecover universal to all TOE Probes	Seamless and very elastic cover for optimum fit and best patient comfort
Latexfree	Protection of patients sensitive to latex
Efficient applicator	Easy application of cover on the probe
Transmission gel syringe set in place	Fast and simple transmission gel injection, time saving
Individually tested	Optimum safety for patient and staff



RECOMMENDED COVERING TECHNIQUE



1. Push the transmission gel into the tip of the cover.



2. Remove syringe.



3. Parcel the gel out.



4. Slip the transducer via the applicator into the latexfree cover.



5. Remove applicator.



6. Fasten the cover by means of the twistlock, apply biteguard, and the probe is ready for use.

PACKAGING AND ORDERING INFORMATION

PIP Kit UC 30/15/1000	
Sterile 12 pieces	88511
PU Kit UC 30/16/1000	
Sterile 12 pieces	87111

For more information on Microtek products or to place an order, call **+31 (0) 575 599 200**.

Visit our website: www.microtekmed.com

Ecolab Healthcare
Clean. Safe. Healthy.

Microtek Medical B.V.
A Division of Ecolab

Hekkehorst 24
NL-7207 BN Zutphen

© 2009 Ecolab Inc. All rights reserved.

ECOLAB®