



## Camera drape

Closed system coupler camera drape

# INNOVATIVE. EFFICIENT.

The comprehensive camera drape, that allows multiple endoscope exchanges without breaking the sterile field. Single-use, universal to all cameras and endoscopes.



# Camera drape

Closed system coupler camera drape

Our Offer	Your Benefit
Closed camera drape universal to all cameras and endoscopes	Cost effective, convenient
Fast, efficient endoscope exchanges during e.g. urology procedures without breaking the sterile field	Increases patients safety, time saving
No pre-surgical soaks for endoscope cameras necessary	Reduction of repair costs, time saving
Lens treated with anti-fogging agent	Optically clear lens
New Transfer-Stick	Easy aseptic transfer of drape to the camera
Latex-free ISO-Silk™ material	Soft pliable material reduces the potential for pinholes



## PACKAGING AND ORDERING INFORMATION

### Closed System Coupler Camera Drape

9907

7" × 97" (18 cm × 246 cm)

Universal to all cameras and endoscopes

Sterile Single-use Packaging:  
1 per package / 10 per case

For more information on  
Microtek products or to place an  
order, call +31(0) 575 599 200 (INT)  
or +44 (0) 1782 561911 (UK).

Visit our website: [www.microtekmed.com](http://www.microtekmed.com)

## RECOMMENDED DRAPING TECHNIQUE



Scrub nurse inserts hand inside drape and holds Transfer-Stick for preparation of camera attachment to coupler.



While the scrub nurse holds Transfer-Stick, circulating nurse attaches camera to coupler.



Double check security of camera attachment. Camera release should be placed perpendicular to release on coupler drape (prevents accidental detachment during use).



Circulating nurse unfolds the telescopically folded drape by grasping the pull ties and pulling the drape over the camera and power cord.



Scrub nurse removes Transfer-Stick and discards.



Endoscope can now be placed inside coupler for surgical use. Endoscope can be changed as needed by releasing locking device on coupler drape.

**Ecolab Healthcare**  
Clean. Safe. Healthy.

**Microtek Medical B.V.**  
A Division of Ecolab  
Hekkehorst 24  
NL-7207 BN Zutphen

© 2008 Ecolab Inc. All rights reserved.

**Ecolab Europe GmbH**  
Richtistr. 7  
CH-8304 Wallisellen  
[www.ecolab.com](http://www.ecolab.com)

**ECOLAB®**